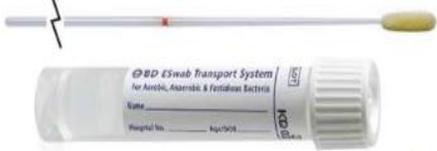


Orderable Test/s	Collection Container (Preferred)	Collection Container (Acceptable/ Alternative)	NOT Acceptable/ To be Rejected
<p>Culture, Body Fluid (Aerobic and Anaerobic) w Gram Stain (LAB269)</p> 	<ul style="list-style-type: none"> Anaerobe Systems Transport Medium (SAP 13231) 	<ul style="list-style-type: none"> Sterile container 	<ul style="list-style-type: none"> EDTA Red top vacutainer Formalin Eswab BBL culture swab Sterile swabs 
<p>Culture, Tissue (Aerobic and Anaerobic) w Gram Stain (LAB898)</p> 	<ul style="list-style-type: none"> Anaerobe Systems Transport Medium (SAP 13231) 	<ul style="list-style-type: none"> Sterile container 	<ul style="list-style-type: none"> EDTA Red top vacutainer Formalin Blood culture bottles Eswab BBL culture swab Sterile swabs 
<p>Culture, Aerobes and Anaerobes, Other w Gram Stain (LAB1291)</p> 	<ul style="list-style-type: none"> Anaerobe Systems Transport Medium (SAP 13231) 	<ul style="list-style-type: none"> Sterile container Eswab (SAP 353663) 	<ul style="list-style-type: none"> Sterile swab BBL culture swab

Orderable Test/s	Collection Container (Preferred)	Collection Container (Acceptable/ Alternative)	NOT Acceptable/ To be Rejected
<p>Culture, Aerobes Only, Other w Gram Stain (LAB897)</p> 	<ul style="list-style-type: none"> • Sterile container 	<ul style="list-style-type: none"> • ESwab (SAP 353663) 	<ul style="list-style-type: none"> • Sterile swab • BBL culture swab
<p>Culture, Fungus (LAB6222)</p> 	<ul style="list-style-type: none"> • Anaerobe Systems Transport Medium (SAP 13231) • Sterile container 	<ul style="list-style-type: none"> • Eswab (SAP 353663) 	<ul style="list-style-type: none"> • Sterile swab • BBL culture swab
<p>Culture, Mycobacteria, Non-blood (LAB 877)</p> 	<p>Anaerobe Systems Transport Medium (SAP 13231)</p>  <p>Sterile container</p>	<p>Eswab (SAP 353663) (Individual swab collection for this test required)</p> 	<ul style="list-style-type: none"> • Sterile swab • BBL culture swab